



Surgitip Aspirator tips:
Surgitip
2,5 mm or 4,8 mm tip opening
Surgitip-micro
1,2 mm tip opening
Surgitip-endo
0,35 mm tip opening

New member of the strong aspirating trio:
Surgitip-endo

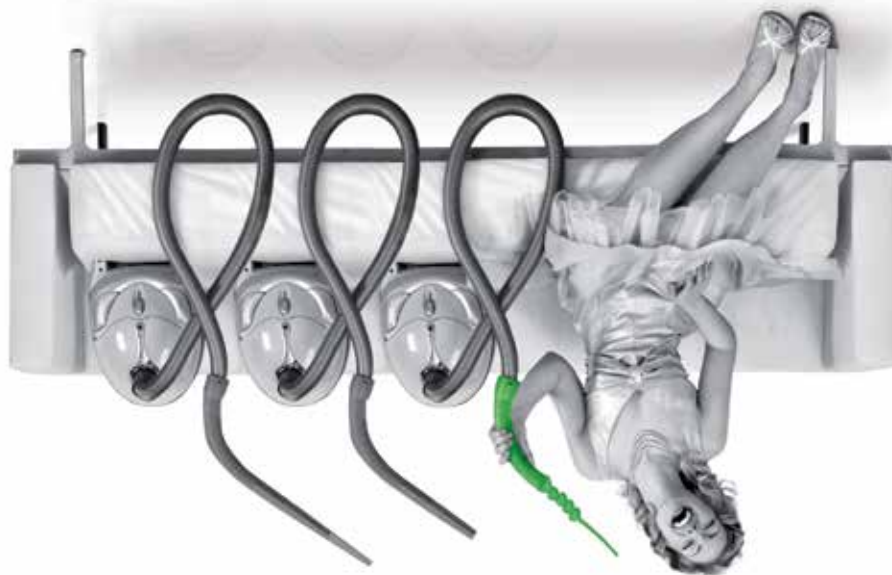

SURGITIP-ENDO ASPIRATOR TIPS

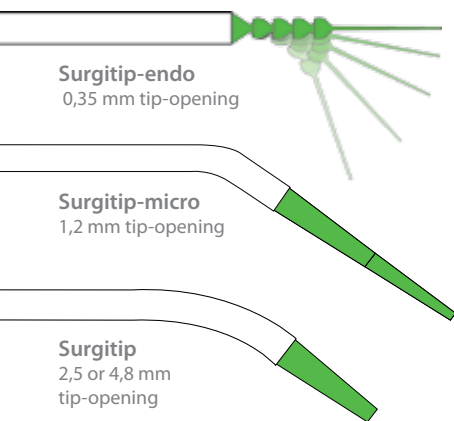

Surgitip-endo
0,35 mm tip-opening

Surgitip-micro
1,2 mm tip-opening

Surgitip
2,5 or 4,8 mm
tip-opening



The Route to successful endodontics:
Scan the QR code
and see our full endo
catalogue

My Surgitip order

Description	Content	REF	Order quantity
Surgitip-endo	20 pc.	6001 8280	
Surgitip-micro	20 pc.	462 012	
	100 pc.	462 013	
Surgitip 2,5 mm	20 pc.	462 025	
	100 pc.	462 026	
Surgitip 4,8 mm	20 pc.	462 048	
	100 pc.	462 049	
Double Adapter	1 pc.	463 000	

What system are you currently using?



Name

Practice

Practice stamp

Street / Number

Town /Postcode

e-mail

Dealer / Town

I agree that Coltène/Whaledent sends the latest offers
and information to the given email-address.

You can unsubscribe at any time. Your data will not be
forwarded to third parties at any time.

Fax the order form

Coltène / Whaledent Ltd., Burgess Hill,
United Kingdom

Fax: +44 (0)14 44 / 870 -640

Coltène / Whaledent GmbH + Co. KG,
Langenau /Germany

Fax : +49 (0) 73 45 / 805 -206



Coltène/Whaledent GmbH + Co. KG
Raiffeisenstraße 30
89129 Langenau/Germany
Tel. +49 (0)7345 805 0 • Fax - 201
www.coltene.com

Coltène / Whaledent Ltd.
The President Suite-A, Kendal House,
Burgess Hill, Victoria WayWest Sussex,
RH15 9NF / U.K.
Fon +44 (0)14 44 23 54 86
Fax -87 06 40

Surgitip-endo

Endodontic aspirator tip for the root canal

- Specially designed tip for multi-directional flexibility
- Effective aspiration and good canal accessibility
- Good visibility of working area
- Manufactured in clean room to EN ISO 14644
- Canal Tip ISO 60, inner diameter 0.35 mm

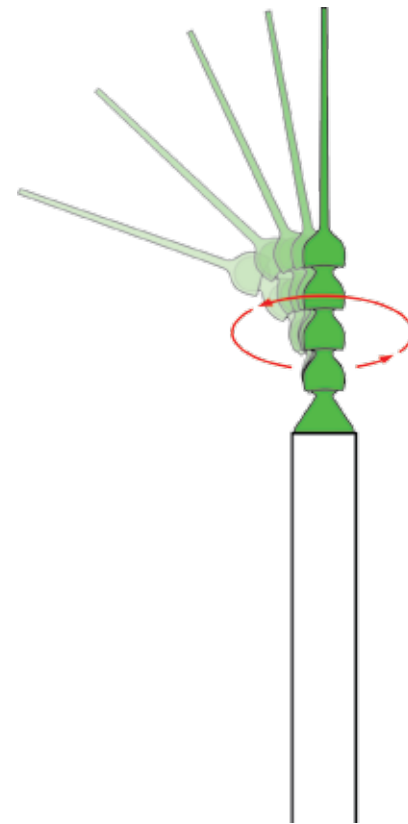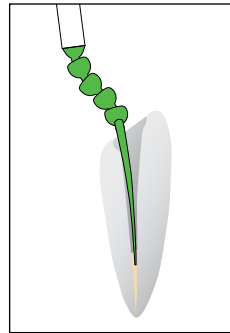

roeko Surgitip-endo



Handling Instructions

- Introduce the tip vertically into the canal
- Use slight tapping movements during aspiration
- Rinse the aspirator in between with water to prevent clogging
- Tissue and/or larger particles can be removed as usual by irrigation and aspiration with the aid of Surgitip-micro or Surgitip.
- For final drying use a ROEKO/HYGENIC Paper Point



A Double Adapter for both 11mm and 16mm is included in every package. If you require one for testing please contact us on Freephone 0500 295454 or your local Sales Representative.